




CEMENTED POLYETHYLENE CUP


GOLD STANDARD

TECHNIQUE OPÉRATOIRE


1/ CUP REAMING

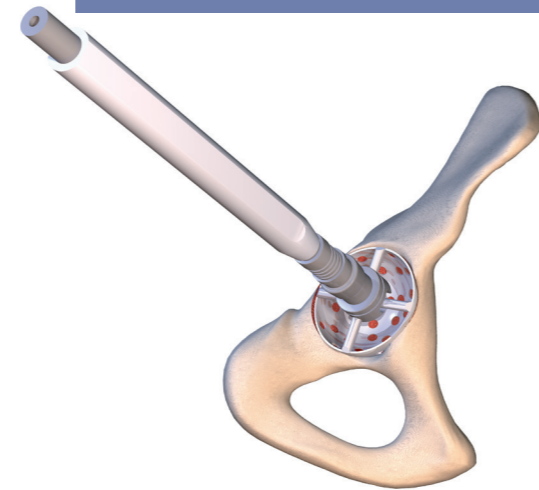

2/ TRIAL

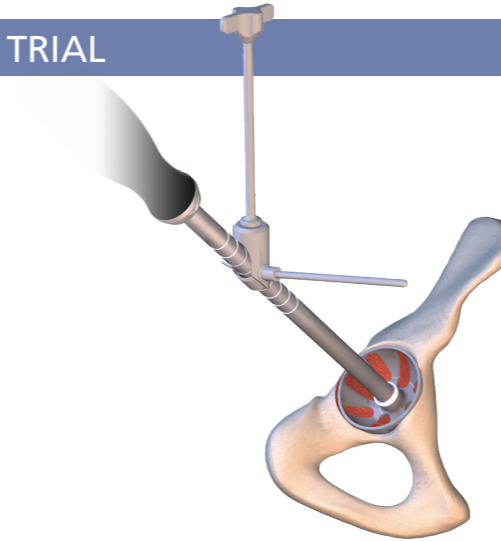

3/ INSERTION OF THE IMPLANT

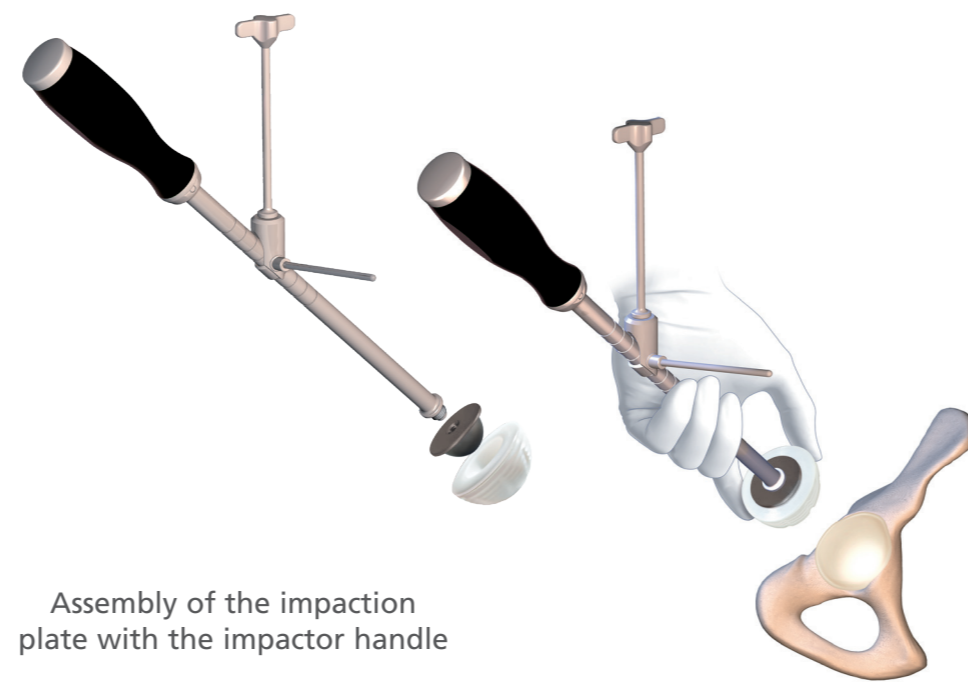

Assembly of the impactation plate with the impactor handle

Insertion of CERAFIX HV cement in the acetabulum and impaction of the PE Cup



Removal of excess cement around the PE cup

CEMENTED POLYETHYLENE CUP



Implant & Surgical Technique

Ø Int. (mm) \ Ø Ext. (mm)	44	46	48	50	52	54	56	58	60	62	64
22,2	747	748									
28	650	835	836	837	484	838	496	839	791	793*	112*
32			593	358	483	359	495	360	792	794	110

Usual sizes

* Parts Items available on request



Nomenclature 10433
TRAY 1

Reference	Description
581	STRAIGHT HANDLE FOR CEMENTED CUPS*
5793	STRAIGHT HANDLE FOR CEMENTLESS CUPS*
679	TRIAL CUP Ø 44
680	TRIAL CUP Ø 46
681	TRIAL CUP Ø 48
682	TRIAL CUP Ø 50
323	TRIAL CUP Ø 52
683	TRIAL CUP Ø 54
324	TRIAL CUP Ø 56
684	TRIAL CUP Ø 58
325	TRIAL CUP Ø 60
326	TRIAL CUP Ø 62
1168	INSERT IMPACTOR Ø 28 MM
689	INSERT IMPACTOR Ø 32 MM
2170	PE CEMENTED CUP DECK IMPACTOR INT. 28 MM
2175	PE CEMENTED CUP DECK IMPACTOR INT. 32 MM
4768	ALUMINA INSERT IMPACTOR Ø 28 MM
4769	ALUMINUM INSERT IMPACTOR Ø 32 MM



Nomenclature 10433
TRAY 2

Reference	Description
366	REAMER Ø 44
367	REAMER Ø 46
368	REAMER Ø 48
369	REAMER Ø 50
370	REAMER Ø 52
371	REAMER Ø 54
372	REAMER Ø 56
373	REAMER Ø 58
374	REAMER Ø 60
375	REAMER Ø 62
364	REAMER HANDLE
1712	MANUAL HANDLE
2128	ADAPTER FOR MOTOR

Reference	CERAVER cements range
646	CERAFIX HV
789	CERAFIXGENTA BV (with gentamicin)
647	CERAFIXGENTA HV (with gentamicin)

BV: Low viscosity - HV: High viscosity

*Available in a curved impactor handle version: nomenclature 5771

Please contact CERAVER for any request concerning the instrumentations of the acetabular cup range internal diameter 22.2 mm and external diameter 64 mm.

Document intended for the exclusive use of healthcare professionals. Cemented PE Cup® - hip prosthesis - is a class III CE marked medical device made by CERAVER - LES LABORATOIRES OSTEAL MEDICAL Company and for which Conformity assessment was carried out by Notified Body G-MED n°0459. Cemented PE Cup prosthesis is intended to replace completely a hip joint that cannot be treated through other therapies. Before any surgical procedure, read carefully instructions for use and surgical technique. For proper use and installation of these devices, qualified professionals must use instruments of the associated kit. Cemented PE Cup is a medical device covered by Health Insurance under certain conditions. Consult the terms and conditions on ameli.fr website.



CERAVER - LES LABORATOIRES OSTEAL MEDICAL
69, RUE DE LA BELLE ÉTOILE
95957 ROISSY CDG CEDEX - FRANCE
TEL. : (+33) 1 48 63 88 63 - FAX : (+33) 1 48 63 88 99
WWW.CERAVER.COM - CONTACT@CERAVER.COM

CERAVER OSTEAL AMERICA INC.
2277 GUENETTE
SAINT LAURENT QUEBEC
H4R 2E9 CANADA
TEL: 19 1 514 856 24 35
FAX: 19 1 514 856 24 44
coal@qc.aira.com

CERAVER IBERICA
BELTRAN BAGUENA, 5-10-10
46009 VALENCIA
ESPAÑA
TEL: +34 96 348 10 17
FAX: +34 96 348 10 02
info@ceraver.es

CERAVER ITALIA
VIA PRATI, 29
40012 CALDERARA DI RENO (BO).
ITALIA
TEL: +39 051 72 88 76
FAX: +39 051 72 78 03
info@ceraver.it

CERAVER SOUTH AFRICA (PTY) LTD.
11 POLO CRESCENT,
WOODMEAD OFFICE PARK,
WOODMEAD, 2191, SANDTON
GAUTENG - SOUTH AFRICA
TEL: +27-11-656-3719
FAX: +27-865-474-635



0 4 5 9

- FIRST IMPLANTATION IN 1979

- UHMWPE POLYETHYLENE

- GROOVES ON THE EXTERNAL SURFACE
Rotational and tilting stability after cementation

- STAINLESS STEEL RADIOPAQUE MARKER
Radiographic assessment



- RANGE
External diameters: 44 to 64 mm
Internal diameters : 22.2; 28 and 32 mm

